ANI MOC-9 Brochure


CONTROL



The ANI MOC-9 allows the monitoring of the parasympathetic nervous system's tone. It may be used to monitor the balance between analgesia and nociception. The main benefits of using the ANI technology are:

- Predict hemodynamic reactivity
- Helpful to diagnose the etiology of a hemodynamic event
- Refine opioid titration
- Predict post-operative pain



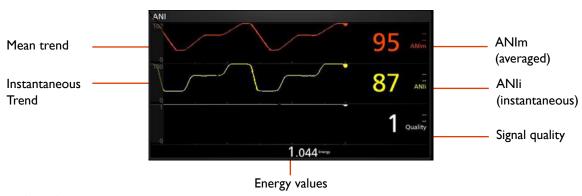

ANI Monitoring

Root patient monitoring and connectivity hub offers plug and play monitoring with Masimo Open Connect™(MOC-9™) modules.

2

A.Mar

Apply sensors on the patient's chest:

- biggest sensor under the right clavicule
- smallest sensor on the cardiac apex



Connect the ANI Sensor VI PLUS to an ANI MOC-9 module



Connect the ANI MOC-9 module to one of three MOC-9 ports on Root

ANI MOC-9 Module Specifications

PHYSICAL CHARACTERISTICS

Length (without cables) Width Thickness Weight (with cables) 155,3 mm 54 mm 22 mm 330 grams

ENVIRONMENTAL

Operating conditions

Temperature at ambient humidity
Humidity
Storage conditions

Temperature at ambient humidity
Humidity

5°C to 40°C 10% to 95%

-20°C to 60°C 0% to 95%

ANI Sensor VI PLUS Specifications

Application site
Usability
Single use
Duration
24 hours max use
Shelf life
Sensor Storage
Others
Others
Chest
Single use
24 hours max use
2 years unopened
0°C to 40°C
Latex free